

Brånemark System® Novum

Titanium Cylinder, Bar, Fixture Level


The Brånemark System® Novum is no longer available in our assortment.

Please find below the Spare Part assortment available for Brånemark System® Novum if there is a need in the future to repair/replace an existing Novum treatment.

Also, attached is the protocol and part numbers to convert a former Novum implant to an existing implant, and for making the corresponding prosthetic solution.

Spare parts assortment

Art. no	Product name
27251	Upper bar Novum
27257	Prosthetic screw Unigrip™ Novum 2-pack
27266	Lower bar screw Unigrip™ Novum 3-pack
27278	Replica Fixture Novum
27279	Replica Lower Bar Novum
27281	Guide sleeve rescue Novum
29780	Gold Cylinder to Fixture Novum
29781	Impression Coping to Fixture Novum

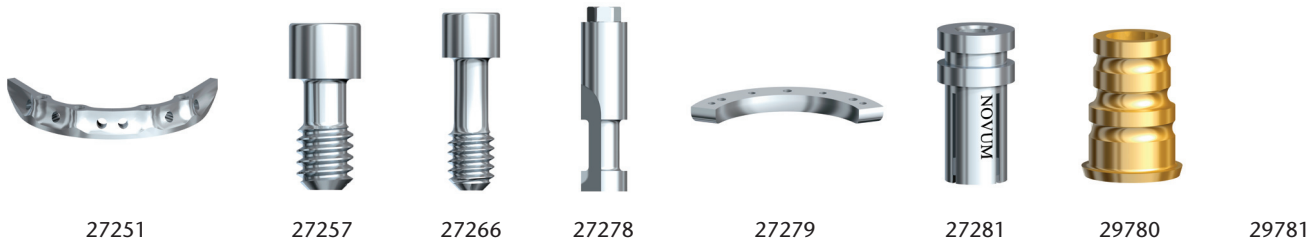

Protocol for adapting Novum implant(s) to a conventional implant treatment

1. Remove the loose Novum implant(s) and supra-structures.
2. Following the healing process, replace the Novum implant(s) with a conventional implant(s) of choice (*note: No Brånemark System® Novum implants are available*).
3. If applicable, place abutments on the conventional implant(s) and connect impression coping(s).
4. Place **Impression Coping to Fixture Novum (#29781)** on the Novum implant(s),
5. Make an impression.
6. Fabricate an implant bridge, using standard components. On the Novum implant(s), use **Gold Cylinder to Fixture Novum (#29780)** and **Replica Fixture Novum (#27278)**.
7. Install bridge to conventional implants following standard procedures. To the Novum implant(s), use **Lower bar screw Unigrip™, 3-pack (#27266)** and tighten to 35 Ncm.